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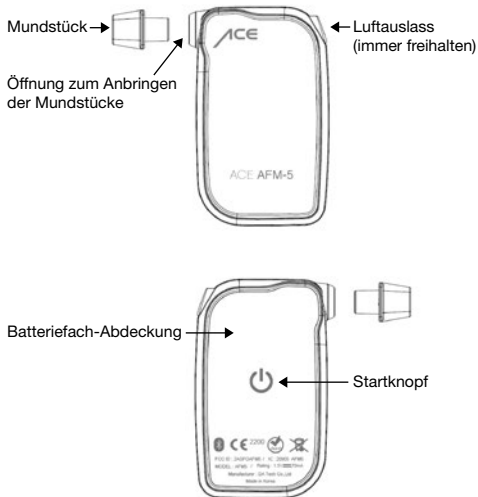
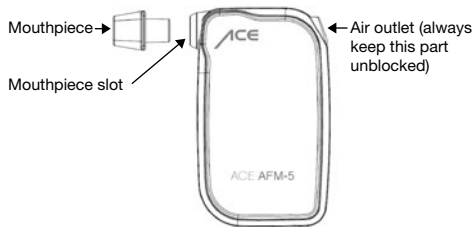
ACE AFM-5 Breathalyzer

ACE
Instruments
PRECISION OR NOTHING.

Quick Start Guide

Label of individual components:

1. Check the batteries and put a mouthpiece into the mouthpiece slot.
2. Download the **"Alcofind"** app from App store (iOS) or Google Play Store (Android), open it and select language and measurement unit (per mill = EU, % BAC = USA, mg/L = UK).
3. Press the **OK** button in the app and check if Bluetooth is switched on on your smartphone.
4. Press and hold the Power button on the breathalyzer until the Blue, Green and Red LEDs appear.
5. The pairing process starts and the Blue LED will blink.
NOTE: In case of the first pairing process, a PIN Code request will be displayed. Type in **"000000"** as PIN Code.
6. If pairing is successful, Green and Red LEDs will appear on the device and the screen on your smartphone will show **"START TEST"**.
7. Tap **"START TEST"** on your smartphone (or press Power button on the ACE Instruments AFM-5).
8. Choose a user (in the app).
9. The circular bar on the app indicates the progress of warm-up. When warming up gage reaches 95, take a deep breath and get ready for blowing.
10. When **"Start blowing"** is displayed, blow for 5 seconds into the mouthpiece until the gage reaches 100 (blow steadily and as you would whistle without a break).
11. If a sufficient breath sample is taken correctly, the **"Analyzing data"** will be shown on the app.
12. After a few seconds the estimated blood alcohol concentration will be displayed on the app e.g. in per mill (3 digits, 2 decimals/per mill = setting for EU customers).
13. Tap either **"Retest"** or **"BAC level"** on the app (or press the Power button on the ACE Instruments AFM-5) to test again.

Please note:

- Wait at least 15 minutes after finishing your last drink.
- From hygiene reasons and for accurate measurement results always use a new mouthpiece for each test.